**Item list**

- 1) User Manual
- 2) Control Unit
- 3) Absorgel Pouch
- 4) Power supply (main inlet cables)
- 5) USB Memory Stick
- 6) Timecard
- 7) Padding
- 8) Seal
- 9) Positioner
- 10) Strap
- 11) Pressure Chamber

**Setup of the Pressure Chamber - First time use**

1) Place the positioner grooves onto pressure Chamber ridges. Move forwards or backwards as required.

2) Place the Padding into the Pressure Chamber and insert the Air Vents into the corresponding holes.

3) Attach the fastening band of the Seal into the four studs at the top of the Pressure Chamber.

4) Place the hole in the Seal onto stud at the front of the Pressure Chamber and pull the Seal over the top edge. Ensure the seam of the seal is facing outwards.


5) Place the strap over the seal and attach it to the stud. Wrap the strap around the seal and attach the other end to the same stud.

## Treatment - Regular use

- 1) Connect the Hose from the Pressure Chamber to the Control Unit.
- 2) Plug in external power supply to the Control Unit before plugging it into the wall socket.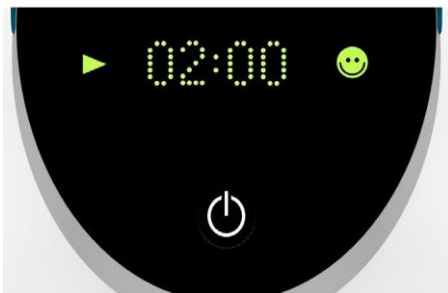

- 7) After the treatment, turn off the device by pressing the on/off button.

- 3) Gently pull the Seal over the outside of the Pressure Chamber.

- 4) Place foot centrally on top of positioner in Pressure Chamber, ensuring air is around the entire leg.

- 5) Pull the seal onto the leg to make the Pressure Chamber airtight.

- 6) Start the treatment by switching on the device by pressing the on/off button.

- 8) Roll down the seal onto the outside of the Pressure Chamber.

- 9) Allow evacuation of the air from the Padding by holding down the release valve button.

- 10) To quickly remove the air, press down on the Padding using your leg or other hand.

**E2 Major Air Leakage**  
Prolonged leakage, treatment is not being delivered. Solution: Check the Seal, Hose with Filter, if the Air vents are still in place and correct if necessary. Restart the device by pressing On/Off Button.

**E3 Low Air Volume**  
Not enough air to function (Low air volume) Solution: check for obstruction, e.g. kinked hose. Restart the device by pressing On/Off Button.

**E4 Data transfer error**  
Data not transferred, e.g. due to taking out USB Memory Stick too early or software error Solution: Switch off the device and try inserting the USB Memory Stick into the Control Unit again.

**Timecard** | When remaining time is 25 hours or less, an hourglass and the time left will show on display Solution: Contact your clinician to load more time with Timecard.

**Note:**  
If the problem persists, contact clinician or local distributor. More information is available in the manual.

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